

ENDOWMENT

- 1x — Unit BE-28UG
- 2x — Wires for electrodes connection
- 2x — Batteries type AA1.5 V (LR6)
- 1x — Instructions booklet
- 1x — Stuffed bag

The vaginal or anal probe, needed for the treatment, will be optionally supplies, within the range PERUPROBE®.



PERINEAL PROBES

Vaginal probe PERIPROBE® VAG 2ST; anatomic shape, 2 ring electrodes gold plated.

Anal probe PERIPROBE® ANA 2ST; anatomic shape, 2 ring electrodes gold plated.

Vaginal probe PERIPROBE® OPTIMA; anatomic shape, 3 dome shape gold plated electrodes. It ensures a better contact with tissues and allows asymmetric treatments, in presence of scars due to episiotomy.

Vaginal probe PERIPROBE® PERISIZE; anatomic shape, 4 dome shape gold plated electrodes. It ensures a better contact with tissues and allows dual channel asymmetric and selective treatments, in presence of scars after episiotomy.

Distributed by:

www.beac.it

TECHNICAL FEATURES

Supply: 2 alkaline batteries 1,5 V AA (LR6)

Output: Max 100 mApp over 1000 Ohm load (using 200 μ S pulse width)

Pulse width: From 50 μ S to 400 μ S adjustable in steps of 50 μ S

Frequency: From 1Hz to 110Hz (1-2-3-4-5-10-12-14-16-18-20-25-30-35-40-45-50-60-70-80-90-100-110)

Wave shape: Biphasic Simmetrical Pulses – Bi-phasic Alternated Pulses

Treatment timer: Continuous, 15min, 30min, 45min, 60min, 90min selectable

ACTION/REST time: From 1 to 40 sec, adjustable in steps of 1 sec.

RAMP time: From 1 to 5 sec, adjustable in steps of 1 sec.

Dimensions: 138mm x 68mm x 28mm

Weight: 160 g. including batteries

WARNINGS!

BE-28UG is a medical device CE. Carefully read the warnings and the instructions for use. The unit cannot be used: - on patients with cardiac pacemakers; - on pregnant women - on areas affected by inflammatory processes.



Produced by: **BEAC BIOMEDICAL**

27040 – Portalbera (PAVIA) ITALY

In CERTIFIED Quality System **EN ISO 9001:2000 – ISO 13485**



IntelliSTIM®

BE-28 UG

Therapeutic unit for the incontinence treatment, through perineal electrical stimulation, suitable to professional or personal use.

“...never been so easy, perineal electrostimulation!”

Therapeutic effectiveness, with no compromises!

IntelliSTIM® BE-28UG




Therapeutic unit for incontinence treatments, through perineal electrical stimulation, suitable to professional or personal use.

Perineal electrical stimulation, through vaginal or anal probes, is one of key treatments in many cases of urinary incontinence. The possibility to extend professional treatments with home care sessions, frequently, make the difference in the therapeutic success. The IntelliSTIM® UG unit, despite provided with sophisticated advanced features and functional flexibility, required for the professional use, it is easy to be used, even directly by the patient.

The advantages of IntelliSTIM® UG unit

Further to the pre-set programs, in memory, the function IntelliSTIM®m which automatically calculate the time based parameters (ACTION / REST), according to the selected frequency and intensity level, allows any professional user, without a deep skill of electro-physiology, building perfect custom programs. Availability of 2 output channels and the possibility to select biphasic pulses or alternated pulses, allow to use any kind of probes.

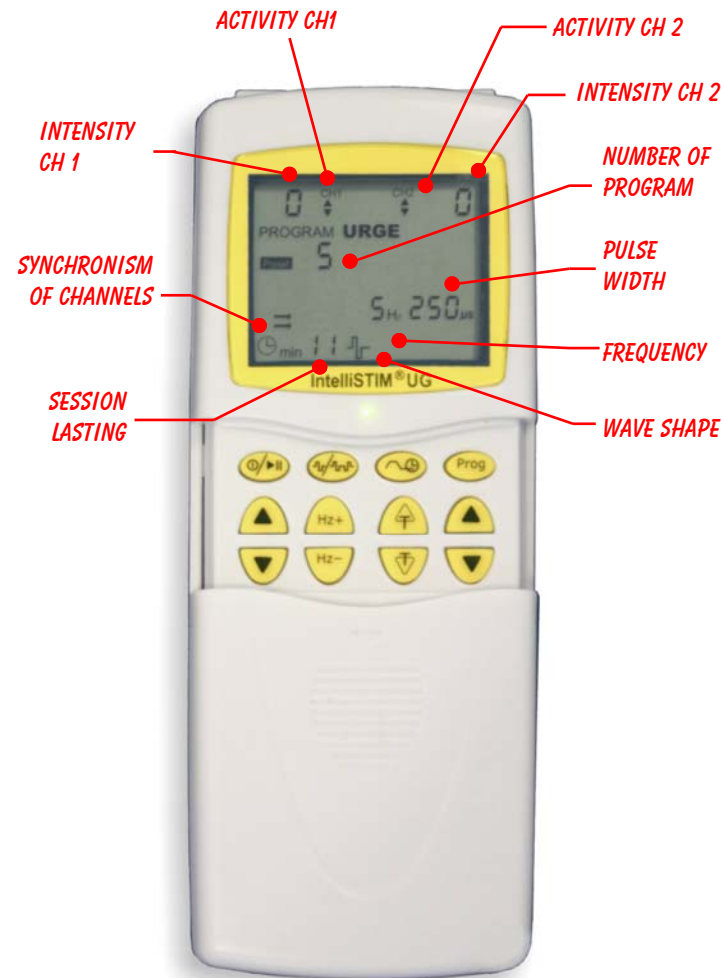

- ◇ Large LCD Display
- ◇ Intuitive operation
- ◇ 4 free programmes (editable)
- ◇ 11 programmes in memory, suitable for different incontinence situations.
- ◇ Special IntelliSTIM® function, with auto-adjustment of time-related parameters
- ◇ Any parameters shown, adjustable in real time, during the session.
- ◇ 2 selectable wave shapes, allowing the optimal use of probes either with ring shape electrodes or lateral electrodes.

- Dual independent channels;
- Last used program loading;
- Count-down timer with switch-off at the end;
- Auto-switch-off if not in use;
- Accumulative recording of the treatment time (in hours);
- Probe protection circuit;
- Lock function capability, by the doctor;